

AWAY FROM BODILY
FUNCTIONS

REDUCES INFECTION
RISK AND FACILITATE
INFECTION CONTROL

DOES NOT INTERFERE
WITH PRESSURE
RELIEF OF MATTRESS

MORE PATIENT
COMFORT

DOES NOT REQUIRE
TO BE REPOSITIONED
DAILY

LESS HANDLING, LESS
BREAKAGE

SOLVENT WIPE-
DISINFECTANT

EASY TO MOVE FROM
ROOM TO ROOM

INTEGRATES TO MOST
NURSE CALL SYSTEMS

WIRED & WIRELESS

WARRANTY- 12 MTHS

Under Mattress Sensor Mat- WIRED



MED- B819-RJ11 (with RJ11)

MED- B819-JS63 (with Jack 6.35mm)

Indigo Care provides a pressure sensitive Premium Under Mattress Sensor Mat that is designed to immediately alert staff should a resident leave their bed. This is most useful to residents who are at risk of falling out of their bed, or those who require assistance in arising from the bed. The mat is placed underneath the mattress, directly under the buttock area. For a normally open call system, a monitor is connected to the bed sensor mat and then the mat is connected to your existing nurse call system with a Y Cable (via the monitor). Once the resident removes themselves from the bed the sensor mat will activate the monitor's alarm and alert the staff. Set-up instructions are printed on the part. Eyelets at the corner of the pad are used to strap the mat to a fixed position.

ADVANTAGES OF UNDER MATTRESS SENSOR MAT

- Away of bodily functions, therefore, more comfort to the user;
- Pressure relief function of mattress is not compromised;
- Reduces infection risk and improve management & handling of infection control;
- No more need to reposition mat multiple times a day;
- Less handling means less breakage

Note: Under Mattress Sensor Mat may not work effectively with all the alternating air mattresses. Please check before purchase.

KEY SPECIFICATIONS:

- **Dimensions: 840mm X 300mm**
- **Connection: RJ11 for normally open; Jack 6.35mm plug for normally close**
- **Electrical resistance < 500 Ohm**
- **White PVC, Thickness 0.20mm**
- **Warranty: 12 months**

PRODUCT ACCESSORIES:

- **RJ11 requires Y Cable**
- **Jack 6.3 requires straight 6.35mm or 3.5mm cable**
- **Anti-Tamper Monitor (use 4 X AA batteries-not included)**
- **6V DC Adaptor (optional)**

Horizon Electronic Components T/as INDIGO CARE
2/71 Rusdale Street, Knoxfield, VIC, 3180
PH +61 3 9763-2988 FAX +61 3 9763-3811
www.indigocare.com.au
ABN 36007302801



ORDER INFORMATION

Fax or email your purchase orders to
03-9763-3811
Or email sales@indigocare.com.au

