

Wireless Suspense Call Point

(Battery Operated)

Indigo Care has designed a Wireless Suspense Call Point which can suspend all the wireless devices in the room. This device deactivates all wireless sensor mats and neck/wrist pendants temporarily so that the care-givers can monitor the residents and/or perform general housekeeping with ease, without setting off the nurse call system.

The call point is designed with 2 buttons: a green PRESENCE and a white CANCEL. Needing only a Lithium Coin Battery (CR 2032) to power, it can be installed anywhere, but ideally next to the door entrance. Simply use the included screw kit or double sided tape if preferred (not included).

Programming the call point to the RRU (Indigo Care's Wireless Receiver) is easy but must be done separately for PRESENCE and CANCEL. See our [Wireless Programming Instructions](#) for complete details.

When the care-givers need to enter the room and suspend the RTU (Indigo Care's Wireless Transmitter), simply press the PRESENCE button. The button will light-up momentarily. This will suspend all RTU devices in the room for 5 minutes (factory default setting).

You will notice that once suspended, the LED light on the RRU will change from a green continuous light to a blinking amber light. This is an indication that the RTU devices are in suspension mode.

Upon approaching the end of the 5 minute suspension, the RRU will give you a 10 second buzz warning. If you require more time, simply press the PRESENCE button again, and the 5 minute cycle starts over.

If you leave the room before the 5 minutes lapses, press the CANCEL button upon your departure. Alternatively, it will automatically reset after 5 minutes.

At the end of the suspended time, the RRU will have a continuous green light, indicating that every RTU device in the room has been reset and is again active.

Note: The Suspense Call Point does *NOT* de-activate the wired pendant or other types of wired devices connected directly to the Nurse Call Point.



Key Specifications:

- Australian Design
- C-Tick Approved
- Anti-Bacterial Silicon Buttons
- Raised, Easy to Press Buttons
- 2 Buttons: Green PRESENCE, White CANCEL
- ID Encoded RF Transmitter
- Momentary Back Light
- Input Supply: Lithium Coin Battery CR 2032
- Screw Kit included
- Optional Double Sided Tape
- Suits Standard Australian Electrical Boxes and Mounting Enclosures

Part number: MED-RTU-PRES



HORIZON ELECTRONIC COMPONENTS Pty Ltd
2/71 Rushdale Street, Knoxfield, VIC, 3180
PH +61 3 9763-2988 FAX +61 3 9763-3811
www.horizone.com.au
ABN 36007302801

05/20



ORDER INFORMATION

Fax or email your purchase orders to
03-9763-3811

Or email sales@horizone.com.au